

# VANISHPOINT®

## Safety by design

Safety mechanism is built into the tube holder

Studies show that most needlestick injuries occur within seconds after needles are removed from patients\*

Safety mechanism is activated BEFORE needle is removed, effectively reducing exposure to contaminated sharps

Needle retracts directly from patient into tube holder, protecting users from both ends of contaminated needle

Prevents recapping and needle removal

Single-use holder complies with safety regulations and prevents cross contamination

### Safe for you and your patients

Standardize your safety system with VanishPoint® syringes



\* CDC (1997). Evaluation of safety devices for preventing percutaneous injuries among health-care workers during phlebotomy procedures—Minneapolis-St. Paul, New York City, and San Francisco, 1993-1995. MMWR 46(2):21-25.

Catalog Number Description

<b>VanishPoint® Allergy Syringe Tray</b>	<i>25 per tray/1000 per case</i>
10134	1ml VanishPoint allergy syringes 27G x 1/2"
<b>1ml VanishPoint® syringes</b>	<i>100 per box/800 per case</i>
10131	1ml VanishPoint tuberculin syringe 27G x 1/2"
10151	1ml VanishPoint tuberculin syringe 25G x 5/8"
10161	1ml VanishPoint tuberculin syringe 25G x 1"
10211	1ml VanishPoint U-100 insulin syringe 29G x 1/2"
<b>3ml VanishPoint® syringes</b>	<i>100 per box/600 per case</i>
10301	3ml VanishPoint syringe 25G x 5/8"
10391	3ml VanishPoint syringe 25G x 1"
10311	3ml VanishPoint syringe 23G x 1"
10321	3ml VanishPoint syringe 23G x 1 1/2"
10331	3ml VanishPoint syringe 22G x 1"
10341	3ml VanishPoint syringe 22G x 1 1/2"
10351	3ml VanishPoint syringe 21G x 1"
10361	3ml VanishPoint syringe 21G x 1 1/2"
10371	3ml VanishPoint syringe 20G x 1"
10381	3ml VanishPoint syringe 20G x 1 1/2"
<b>5ml VanishPoint® syringes</b>	<i>100 per box/600 per case</i>
10531	5ml VanishPoint syringe 22G x 1"
10541	5ml VanishPoint syringe 22G x 1 1/2"
10551	5ml VanishPoint syringe 21G x 1"
10561	5ml VanishPoint syringe 21G x 1 1/2"
10571	5ml VanishPoint syringe 20G x 1"
10581	5ml VanishPoint syringe 20G x 1 1/2"
<b>10ml VanishPoint® syringes</b>	<i>100 per box/600 per case</i>
11031	10ml VanishPoint syringe 22G x 1"
11041	10ml VanishPoint syringe 22G x 1 1/2"
11051	10ml VanishPoint syringe 21G x 1"
11061	10ml VanishPoint syringe 21G x 1 1/2"
11071	10ml VanishPoint syringe 20G x 1"
11081	10ml VanishPoint syringe 20G x 1 1/2"
<b>VanishPoint® blood collection products</b>	
22701	VanishPoint blood collection tube holder <i>250 units per case</i>
22711	VanishPoint small diameter tube adapter <i>25 units per box</i>



**RETRACTABLE TECHNOLOGIES, INC.**

www.vanishpoint.com



© 2006 Retractable Technologies, Inc.  
All rights reserved.  
M006 REV. 4

511 Lobo Lane  
Little Elm, Texas 75068-0009 USA  
Tel: (972) 221-6644  
Fax: (972) 294-4400  
Toll Free: 1-888-703-1010  
domestic: rtisales@vanishpoint.com  
international: rti.intl@vanishpoint.com

## UNCOMPROMISING SAFETY FOR CLINICIANS AND PATIENTS



**VANISHpoint™**  
**BLOOD COLLECTION TUBE HOLDER**

The New Standard for Safety™

1

Securely thread multiple sample blood collection needle into tube holder.



2

Perform venipuncture per protocol of institution.




3

Once last tube has been removed and needle is still in patient's vein, close end cap. Needle automatically retracts out of patient's vein into tube holder.



4

Dispose of VanishPoint® tube holder in an appropriate sharps container.

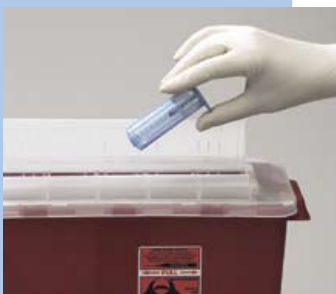


# the VANISHPOINT® ADVANTAGE

Designed for Clinician  
and Patient Safety



End cap is flexible and can be easily closed from any position

Clear, blue tube holder allows easy visualization of tube and blood return

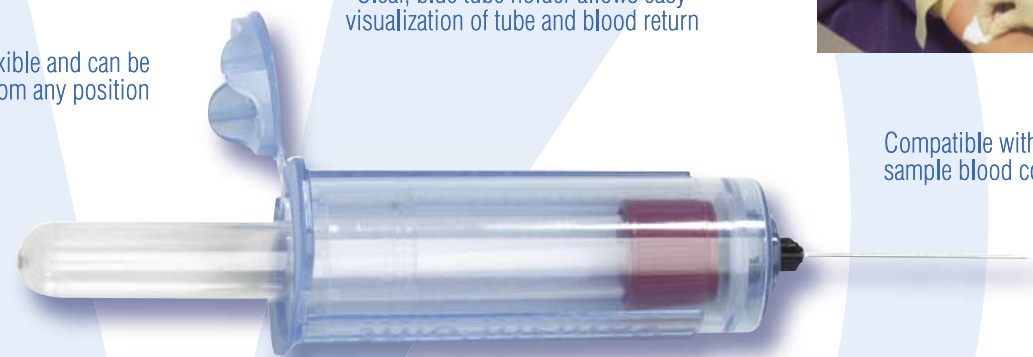

Compatible with standard, multiple sample blood collection needles

Allows users to keep both hands behind needle when activating safety mechanism



Needle is retracted directly from patient virtually eliminating exposure to contaminated needle

Protects users from both ends of contaminated needle