

FOR BEST RESULTS:

Clean application site with an alcohol wipe and/or *non-moisturizing* soap prior to application.


Contents:

- 1 Alcohol Wipe*
- 1 BioPlus+™ Barrier Wipe
- 1 Hydrocolloid Universal Securement Device

For use on intact skin.

If redness or irritation occurs, stop use and consult a healthcare practitioner.

 **CathGrip**®
UniGrip™

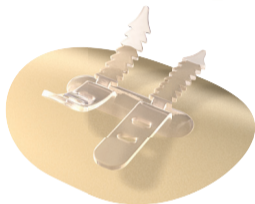

See reverse for complete application instructions.



Manufactured by BioDerm®, inc.

12320 73rd Court N, Largo, FL

Orders: (800) 814-3134 Support: (800) 864-6755

Web: www.BioDermInc.com

**Not included
in part numbers
ending with NA*


Applying UniGrip™




CathGrip®
UniGrip™




STEP 1: Clean entire application site with alcohol wipe and allow to dry.

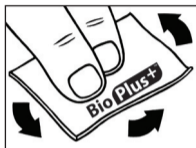

STEP 2: Wipe the entire application site with BioPlus+™. Allow 15+ seconds to dry.

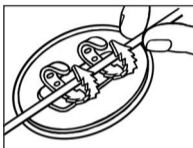

STEP 3: Remove the release papers and press to the skin to seal.

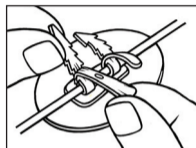

STEP 4: Rest the tube on the soft, no-slip grips. Pull tight to secure.

ATTENTION: Following application, leave a one inch space around the CathGrip® device when using any cleansing products, creams or lotions. Contact with the hydrocolloid will result in early dislodgement of the securement device.