


Drop-It[®] Quick Release Needle Holder

May be used with standard blood collection devices.
Suggested use with the **Punctu-Guard** Blood Collection Needle.

- Internal blunting technology which is integral to the needle itself.
- Blunts while in the patient's vein, eliminating exposure during the first few critical moments.
- Reduced needle sticks by 76%, the highest rate of any device studied.

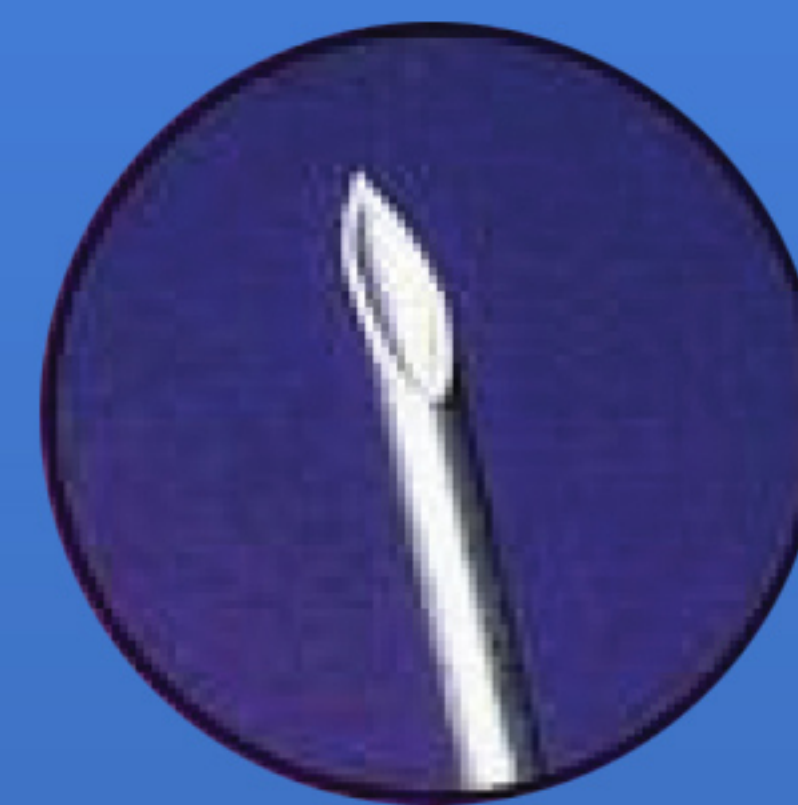

Sharp Entry...



Blunt Exit

Recommended for
up to 100 uses!



Drop-It® Quick Release Needle Holder

Ordering Information

Code	Description	Quantity
4550	Drop-It® Quick Release Needle Holders	50 Units per Box

NOTE: During use, exercise care not to depress the needle release button when threading needle onto holder or during procedure.

Always inspect the holder for integrity and cleanliness before each use.

To Disinfect: Soak in 10% bleach solution for 10-20 minutes, then rinse thoroughly. Wash with detergent, rinse and dry. If holder is damaged, discoloured or remains soiled after disinfection, discard.

Maximum 20 cleanings per holder. DO NOT soak overnight.



CardioMed Supplies Inc.

199 Saint David Street
Lindsay, Ontario, Canada
K9V5K7

www.cardiomed.com

Tel: 1-705-328-2518

Fax: 1-705-328-9747

Toll-free: 1-800-387-9757

mail@cardiomed.com