



Instantly removes hydrocolloid, dressing tapes and other adhesives without irritation. Flash dries in 15 seconds without leaving a residue which allows for immediate reapplication.


## HOW TO USE



### Step 1


Hold foil packet vertically, tear at notch to open and remove cloth from packet. Keep cloth folded to retain maximum saturation.

- ! Keep the packet held upright to avoid
- spilling the excess FreeDerm solution.

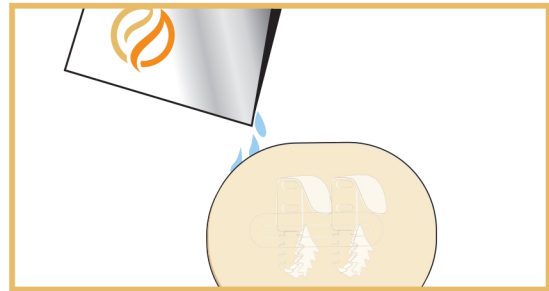


### Step 2

Drip the excess FreeDerm from the packet onto the top edge of the dressing, allowing the solution to drip downwards to start releasing the adhesive bond between the skin and dressing.

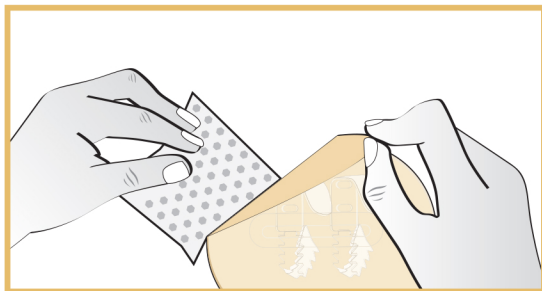

### Step 3

Once the dressing edge is lifted, use the folded cloth to wipe underneath the dressing as you gently pull the dressing off.

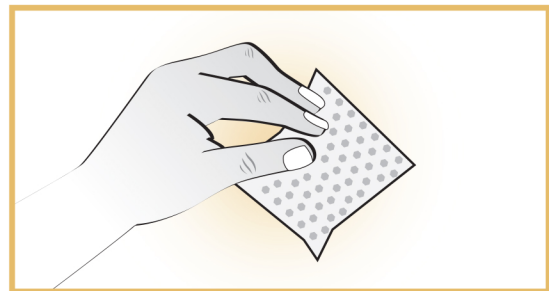

### Step 4

Once the dressing is completely removed, wipe the area to remove any residue adhesive.

If redness or irritation occurs, discontinue use and consult your healthcare professional. Avoid contact with eyes and mouth. For external use only. Avoid inhalation.

©2021 BioDerm, Inc. All rights reserved. BioDerm® and FreeDerm® are registered trademarks of BioDerm, Inc.



CardioMed Supplies Inc. / Nipro Canada  
199 Saint David St.  
Lindsay, Ontario  
K9V 5K8 Canada  
T: 705-328-2518  
service@cardiomed.com  
www.cardiomed.com

LT 70049 IR